# Bittium

---

## MedicalSuite™

## HSAT-

## Serviceplattform

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---

Bittium Biosignals Ltd.  
Pioneerinkatu 6  
70800 Kuopio  
Finnland  
Telefon: +358 40 344 2000  
www.bittium.com

## Rechtsvermerk

---

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Bittium									
Dashboard			Recordings			EN - Sales Technician Technician			
Recordings									
Q Search		Create recording		10		All			
Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
KK02	Active	28.10.2021 15:25	28.10.2021 15:46	Sleep sales center	28.10.2021 15:23	Bittium HSAT	0000568	Sales diag group 1	Download
Pat123456	Closed	19.10.2021 11:41	19.10.2021 12:12	Sleep sales center	19.10.2021 11:40	Bittium HSAT	0000673	Sales diag group 1	Download
PID0000b	Closed	18.10.2021 17:48	18.10.2021 18:16	Sleep sales center	18.10.2021 17:47	Bittium HSAT	0000673	Sales diag group 1	Download
PID0000a	Closed	18.10.2021 16:19	18.10.2021 16:15	Sleep sales center	18.10.2021 16:18	Bittium HSAT	0000536	Sales diag group 1	Download
PID00009	Active	16.10.2021 08:30	09.11.2021 10:16	Sleep sales center	16.10.2021 08:29	Bittium HSAT	0000536	Sales diag group 1	Download
PID00008	Active	16.10.2021 08:25	16.10.2021 09:04	Sleep sales center	16.10.2021 08:25	Bittium HSAT	0000568	Sales diag group 1	Download
Test008	Closed	15.10.2021 19:01	18.10.2021 15:26	Sleep sales center	15.10.2021 19:00	Bittium HSAT	0000673	Sales diag group 1	Download
TEST007	Closed	15.10.2021 18:46	15.10.2021 19:14	Sleep sales center	15.10.2021 18:43	Bittium HSAT	0000673	Sales diag group 1	Download
PID00007	Active	No date	14.10.2021 17:22	Sleep sales center	14.10.2021 17:18	Bittium HSAT	1234567	Sales diag group 1	Download
23_45_08	Closed	01.10.2021 16:17	14.10.2021 17:19	Sleep sales center	01.10.2021 16:16	Bittium HSAT	0123456	Sales diag group 1	Download

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The screenshot shows the 'Recordings' section of the Bittium interface. At the top, there are navigation tabs for 'Dashboard' and 'Recordings', along with user information 'EN' and 'Technician 1'. Below the navigation is a search bar and a 'Create recording' button. The main area contains a table with columns: Patient ID, Status, Last data transfer, Last update, Organization, Created, Type, Device, Assignment, and Action. A dropdown menu is open for the 'Assignment' column of the first row, showing a search bar and a list of groups: 'Cardioscope Group', 'Diagnostic group', 'Diagnostic group (autoreports)', 'HRV group (autoreports)', and 'HSAT Analysis' (which is selected). Below the groups, it lists 'Users in HSAT Analysis' with 'Dr. Hanna HSAT'.

Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
testPDF	Closed	23.01.2024 15:14	25.01.2024 13:41	ServiceProvider	23.01.2024 15:14	Bittium	0000533	HSAT Analysis	[Download]
HSAT_ECG	Ended	16.01.2024 16:13	16.01.2024 16:17	DemoClinic1	16.01.2024 16:13	Bittium			[Download]
HSAT_2_RIP_Belts	Active	16.01.2024 16:11	25.01.2024 10:10	DemoClinic1	16.01.2024 16:11	Bittium			[Download]
PID00314	Ended	14.12.2023 15:06	19.12.2023 13:09	DemoClinic1	14.12.2023 15:05	Bittium			[Download]
PID002f3	Closed	22.11.2023 11:37	22.11.2023 11:41	DemoClinic1	22.11.2023 11:37	Bittium			[Download]

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The screenshot shows the 'Recordings' section of the Bittium interface. At the top, there are navigation tabs for 'Dashboard' and 'Recordings', along with user information 'EN' and 'Sales Technician'. Below the navigation is a search bar and a 'Create recording' button. The main area contains a table with columns: Patient ID, Status, Last data transfer, Last update, Organization, Created, Type, Device, Assignment, and Action. A dropdown menu is open for the 'Action' column of the first row, showing a search bar and a list of items: 'Recording data', 'Recording data 1 day(s)', 'Reports', '6169a6099ec5e8000acd5a02\_statement.pdf', and 'Other documents', 'recording-information.pdf'.

Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
Test008	Closed	15.10.2021 19:01	18.10.2021 15:26	Sleep sales center	15.10.2021 19:00	Bittium HSAT	0000533	Sales diag group 1	[Download]
TEST007	Closed	15.10.2021 18:46	15.10.2021 19:14	Sleep sales center	15.10.2021 18:43	Bittium HSAT	0000533		[Download]
PID0000b	Closed	18.10.2021 17:48	18.10.2021 18:16	Sleep sales center	18.10.2021 17:47	Bittium HSAT	0000533		[Download]
PID0000a	Closed	18.10.2021 16:19	18.10.2021 16:15	Sleep sales center	18.10.2021 16:18	Bittium HSAT	0000533		[Download]

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Organization	Sleep sales center	Created	19.10.2021 11:40	Last data transfer	19.10.2021 11:41
Status	<span style="color: red;">●</span> Closed	Started	18.10.2021 23:30	Last update	19.10.2021 12:12
Assignment	Sales diag group 1 ▾	Ready for analysis	No date	Duration	1 / 1 day(s)

Recording data | **Patient details** | Anamneses | Patient diary | Recording details | Attachments

Cancel Save

### Patient details

Patient ID	Pat123456	Age (years)	55
	<small>Patient ID must be at least 4 characters long and lower/uppercase letters, numbers and underscores are allowed. If Patient ID is left blank the system will generate it (for example PID0000A).</small>	Height (cm)	175
		Weight (kg)	80
		BMI	26.12
		Gender	Female ▾

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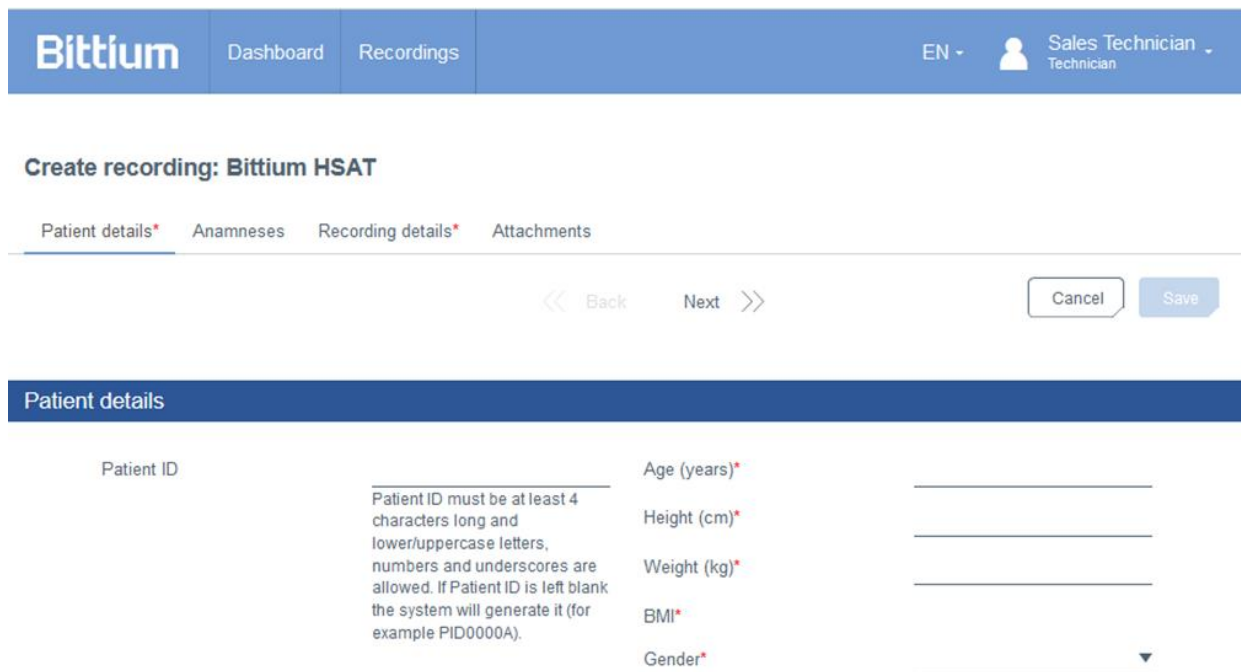
A screenshot of the Bittium MedicalSuite interface. The top navigation bar includes the Bittium logo, 'Dashboard', 'Recordings', and a user profile for 'Sales Technician'. Below this is a header for 'Create recording: Bittium HSAT' with tabs for 'Patient details\*', 'Anamneses', 'Recording details\*', and 'Attachments'. Navigation buttons for 'Back', 'Next', 'Cancel', and 'Save' are visible. The 'Patient details' section contains several input fields: 'Patient ID' with a note that it must be at least 4 characters long and can be auto-generated; 'Age (years)\*', 'Height (cm)\*', 'Weight (kg)\*', 'BMI\*', and 'Gender\*'. Each field has a corresponding input line.

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The screenshot shows the Bittium HSAT recording creation interface. At the top, there is a navigation bar with 'Bittium', 'Dashboard', and 'Recordings'. The user is logged in as 'Sales Technician'.

The main heading is 'Create recording: Bittium HSAT'. Below it, there are tabs for 'Patient details', 'Anamnesen', 'Recording details', and 'Attachments'. The 'Anamnesen' tab is active.

Navigation buttons include '<< Back', 'Next >>', 'Cancel', and 'Save'.

**Medication**

Medication: [Text input field]

Medication during the recording:  No  Yes

[Text input field]

**Medical history**

Close family members diagnosed or treated for a sleep disorder:  No  Yes

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## Chance of dozing off or falling asleep in the following situations

Sitting and reading	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Watching TV	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Sitting, inactive in a public place (e.g. a theatre or a meeting)	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
As a passenger in a car for an hour without a break	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Lying down to rest in the afternoon when circumstances permit	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Sitting and talking to someone	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Sitting quietly after a lunch without alcohol	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
In a car, while stopped for a few minutes in the traffic	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High

## Sleep history

Morning headache	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Dry mouth in the morning	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Daytime fatigue or snoring	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Daytime napping	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Heartburn	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Loss of concentration	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Memory loss	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Cramping at night	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Difficulties falling asleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Breathing breaks while sleeping	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Pain during night	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Snoring or snorting during sleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Waking up during the night or restless sleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
A strong urge to move your feet or an uncomfortable sensation in your feet related to this urge during sleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Urinary urgency during sleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
If yes, how many times per night?	<input type="radio"/> Once	<input type="radio"/> Twice	<input type="radio"/> More often		
Average amount of sleep	_____ Hours				

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## Create recording: Bittium HSAT

Patient details\* Anamneses **Recording details** Attachments

<< Back Next >>

Cancel

Save

### Recording details

Organization	ServiceProvider	Referring physician
Requested report language	English	
Reason for recording		

### Device information

Recorder S/N (21) 19 XXXXXXXX

See the side of the device. **Please add numbers after "19"**.

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## Create recording: Bittium HSAT

Patient details Anamneses Recording details\* Attachments

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Patient ID **PID0a74a** Bittium HSAT

Ready for analysis |

Organization	Sleep test center	Created	29.12.2021 12:08	Last data transfer	29.12.2021 12:20
Status	<span style="color: green;">●</span> Active -	Started	09.09.2021 23:30	Last update	29.12.2021 12:21
Assignment	None -	Ready for analysis	No date	Duration	2 / 2 day(s)

[Recording data](#)
[Patient details](#)
[Anamneses](#)
[Patient diary](#)
[Recording details](#)
[Attachments](#)

### Patient diary - 09.09.2021, 23:30 (06h 59m)

Please, ask the patient to answer the following questions about the recording.

When did you go to bed?	<input type="text" value="hh:mm"/>	In your estimation, when did you fall asleep?	<input type="text" value="hh:mm"/>
What time did you wake up in the morning?	<input type="text" value="hh:mm"/>	In your estimation, how many hours did you sleep?	<input type="text" value="hh:mm"/>
How did you sleep?	<input type="text"/>	Did you get out of bed during the recording?	<input type="text"/>
How did you sleep compared to usual?	<input type="text"/>		

Other comments

If the patient woke up during the recording, please write down the events.

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The screenshot displays the Bittium MedicalSuite interface. At the top, a metadata table shows details for 'Clinic A', including creation and update dates. Below this, a navigation bar highlights 'Recording data'. A 'Download' section is visible with a 'Cancel' button and a 'Save' button. The main area features a table of recording data with columns for Day no., Start time, Duration, Status, Quality, and Notes. Two recordings are listed: one from 10.04.2019 with 'Fair' quality and another from 11.04.2019 with 'Good' quality. A red box highlights the 'Preparing' status icon in the second row. To the right, an 'UPLOADS' dialog box shows the upload progress for two files, with the first file marked as 'Uploaded' and the second as 'Preparing data - 2 / 2'.

Organization	Clinic A	Created	01.04.2019 15:34	Last data transfer	13.04.2019 12:00
Status	Preparing	Started	10.04.2019 12:00	Last update	13.04.2019 12:00
Assignment	None	Ended	No date	Duration	2 days

Day no.	Start time	Duration	Status	Quality	Notes
1 / 2	10.04.2019 22:00	08h 05m	Open	Fair	Add notes
2 / 2	11.04.2019 22:00	07h 50m	Open	Good	Add note

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The screenshot shows the Bittium Respiro Analyst interface. At the top, there is a navigation bar with 'Back', 'Bittium', 'Dashboard', and 'Recordings'. The user is identified as 'Sales Technician' with a language dropdown set to 'EN'. Below this, the patient information is displayed: 'Patient ID: PID0a74a' and 'Bittium HSAT'. A 'Ready for analysis' button is visible. The main content area shows a table with patient details:

Organization	Sleep test center	Created	29.12.2021 11:22	Last data transfer	29.12.2021 11:32
Status	Active	Started	16.12.2021 02:44	Last update	29.12.2021 11:33
Assignment	None	Ready for analysis	No date	Duration	2 / 2 day(s)

Below the table are tabs for 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. A 'Download' button is present, along with a checkbox for 'Include all attachments' and 'Cancel'/'Save' buttons. The 'Recording data' section shows a table with recording details:

Day No.	Start date of file	Duration	Status	Quality	Notes
1/1	16.12.2021 00:44	08h 10m	Open	Good	Add notes

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Download  
 Include all attachments

Cancel Save

<input type="checkbox"/>	Day No.	Start date of file	Duration	Status	Quality	Notes
<input type="checkbox"/>	1/3	14.12.2021 23:42	04h 30m	Done	Fair	Add notes
<input type="checkbox"/>	2/3	14.12.2021 23:42	05h 00m	Open	Good	Add notes
<input type="checkbox"/>	3/3	14.12.2021 23:42	00h 15m	Unusable	Poor	Marked as "Unusable" <ul style="list-style-type: none"><li>Corrupted recording data</li><li>Not enough analyzable data due to noisy signal or unattached recorder</li><li>Not enough sleep to write a statement</li><li>Other (trs)</li></ul>

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Unusable recording data

Patient ID: [patient\_id], 10/04/2019, 22:00 (08h 20m)

Why the recording data of this night is unusable?

Corrupted recording data

Not enough analyzable data due to noisy signal or unattached recorder

Not enough sleep to write the statement

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The screenshot displays the Bittium MedicalSuite interface for a patient recording. At the top, there is a navigation bar with 'Back', 'Bittium', 'Dashboard', and 'Recordings'. The user is identified as 'Sales Technician'. The patient ID is 'PID0a74a' and the recording is for 'Bittium HSAT'. A red box highlights the 'Ready for analysis' button in the top right corner. Below this, a table shows recording details:

Field	Value	Field	Value	Field	Value
Organization	Sleep test clinic A	Created	24.03.2022 15:55	Last data transfer	24.03.2022 15:58
Status	Active (dropdown menu open showing 'Ended')	Started	14.12.2021 23:42	Last update	05.05.2022 13:15
Assignment	None	Ready for analysis	No date	Duration	2 / 2 day(s)

Below the table, there are tabs for 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. The 'Recording data' tab is selected, showing a 'Download' button and a checkbox for 'Include all attachments'. Below this, there is a section for 'Recording data' with a table of recording files:

Day No.	Start date of file	Duration	Status	Quality	Notes
1/2	14.12.2021 23:42	04h 30m	Open	Fair	Add notes

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**Ready for analysis**

Patient ID **PID0a870\_013696**

Are you sure this recording is ready for analysis?

- Please make sure the patient diary is filled or it is attached to "Other documents" section
- You will no longer be able to upload new recording data to it.

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## Diagnostic report needed

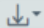
Recording	Assignment	Ready for analysis	Duration	
● PID0a74a	Test diag group 1 ▾	24.11.2021 09:10	1 day(s)	Analyze 

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**Patient ID: PID00298** Bittium HSAT Ready for analysis

<b>Organization</b>	DemoClinic1	<b>Created</b>	30.05.2023 13:56	<b>Last data transfer</b>	30.05.2023 14:01
<b>Status</b>	● Active	<b>Started</b>	23.11.2021 23:42	<b>Last update</b>	25.01.2024 14:08
<b>Assignment</b>	HSAT Analysis	<b>Ready for analysis</b>	No date	<b>Duration</b>	1 / 1 day(s)

Groups

- Cardioscope Group
- Diagnostic group
- Diagnostic group (autoreports)
- HRV group (autoreports)
- ✓ HSAT Analysis

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---

All Not assigned

Recording	Assignment	Ready for analysis	Duration	
● PatientZero	Test diag group 1	24.11.2021 09:10	1 day(s)	<div style="border: 2px solid red; display: inline-block; padding: 2px 5px;">Analyze</div> <span style="margin-left: 10px;">↓</span>

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Summary and statement
Analysis
Report

---

Accept report

1 of 3
Automatic Zoom

**Patient**

Patient ID:	23-30-48	Height:	185cm	Weight:	85kg
BMI:	24.8	Age:	58 years	Gender:	Male

**Summary**

Apnea-hypopnea index: 8.7/h (Supine: 15.4/h, Non-supine: 2.1/h)	Oxygen Desaturation Index (ODI3): 8.7/h
--	---

AHI

ODI3

Recording date	15/10/2021	Recording: TRT	08h 36m (23:30 - 08:07)
Evaluation period	08h 16m (23:43 - 08:00)		

Apnea-hypopnea index (AHI) displayed here is the total number of respiratory events (Apneas + Hypopneas) divided by Evaluation period. Evaluation period was determined based on measured signals and patient diary

**Statement**

**Findings**  
Nearly all occurrences of respiratory events are in the supine position. The recording was technically successful and of high quality.

**Conclusion**  
The recording findings are consistent with mild obstructive sleep apnea. Avoiding sleeping in a supine position (positional therapy), for example by placing a tennis ball in the back of the pajamas or wearing a sleep apnea belt, might be sufficient treatment. It is recommended to confirm the treatment outcome with a new recording.


23/02/2022  
Dr. Sleep Apnea

This report is the consulting doctor's opinion based on the monitoring and the available background information. The final conclusions are always made by the patient's attending doctor who has the best

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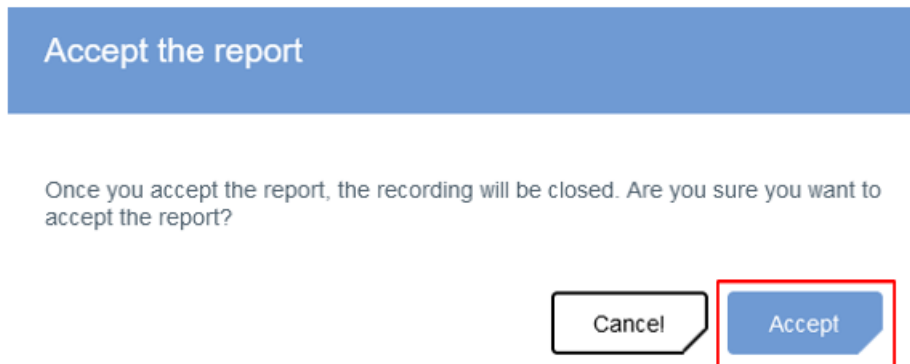Back **Bittium** Dashboard Recordings EN Tiego TechnicianUser Technician

Patient ID: PID0a743\_013381 Bittium HSAT Close

Organization	Sleep test center	Created	13.12.2021 09:29	Last data transfer	13.12.2021 09:31
Status	● Ended ▾ Closed	Started	24.06.2021 00:00	Last update	29.12.2021 16:20
Assignment	Daniela DiagnosticUser ▾	Ready for analysis	13.12.2021 09:44	Duration	1 / 1 day(s)

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**Bittium** Dashboard Recordings EN Sales Technician Technician

Recordings

Create recording ▾ 10 ▾ All ▾

Patient ID ▾	Status ▾	Last data transfer ▾	Last update ▾	Organization ▾	Created ▾	Type ▾	Device ▾	Assignment ▾	Action
PID0a743	● Ended ▾ Closed	13.12.2021 09:31	29.12.2021 16:20	Sleep test center	13.12.2021 09:29	Bittium HSAT	0000552	Daniela DiagnosticUser ▾	

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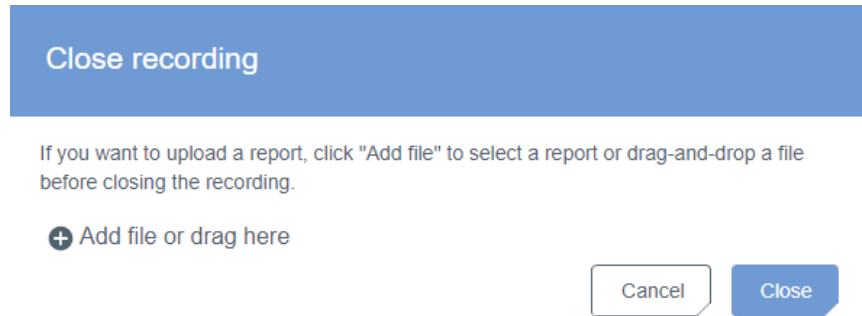
Latest reports				
Recording	Organization	Added	Approval	
● PID0a74a	DemoClinic2	07.10.2020 17:49	07.10.2020 17:54	 

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Recordings

Search: PID0a74a Create recording - 10 - All -

Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
PID0a74a	● Closed	14.12.2021 15:47	17.12.2021 15:22	Sleep test center	14.12.2021 15:47	Bittium HSAT	0123456	None	 

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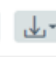
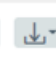
Latest reports				
Recording	Organization	Updated	Approval	
● PID023d3	Sleep test center	16.12.2021 09:46 Not assigned	16.12.2021 09:48	 
● PID0a74a	Sleep test center	15.12.2021 15:16 Not assigned	<span>Approve</span>	 

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The screenshot shows the Bittium interface for a patient with ID PID0a74a. The top navigation bar includes 'Back', 'Bittium', 'Dashboard', 'Recordings', 'EN', and 'Sales Technician'. The patient information section displays 'Patient ID: PID0a74a' and 'Bittium HSAT'. A red box highlights the 'Approve' button in the top right corner. Below this, a table provides details about the recording:

Organization	Sleep test center	Created	14.12.2021 15:47	Last data transfer	14.12.2021 15:47
Status	<span style="color: red;">●</span> Closed	Started	24.02.2021 00:00	Last update	17.12.2021 15:22
Assignment	None	Ready for analysis	14.12.2021 15:49	Duration	1 / 1 day(s)

Below the table, there are tabs for 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. At the bottom, there is a 'Download' button, a checkbox for 'Include all attachments', and 'Cancel' and 'Save' buttons.

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The confirmation dialog box has a blue header with the text 'Confirmation'. Below the header, the text reads: 'Are you sure you want to approve the reports of Patient ID: 01-10-56? After approval, reports will be visible for the clinic.' At the bottom of the dialog, there are two buttons: 'Cancel' and 'Approve'.

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< Back **Bittium** Dashboard Recordings EN Sales Technician Technician

**Patient ID: PID0a74a** Bittium HSAT Approved

Organization	Sleep test center	Created	14.12.2021 15:47	Last data transfer	14.12.2021 15:47
Status	● Closed	Started	24.02.2021 00:00	Last update	31.12.2021 10:13
Assignment	None	Ready for analysis	14.12.2021 15:49	Duration	1 / 1 day(s)

Recording data Patient details Anamneses Patient diary Recording details Attachments

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Latest reports

Recording	Organization	Updated	Approval	
● PID023d3	Sleep test center	16.12.2021 09:46 Not assigned	16.12.2021 09:48	<span style="border: 1px solid red; padding: 2px;">📄</span> <span>↓</span>
● PID0a74a	Sleep test center	15.12.2021 15:16 Not assigned	16.12.2021 09:48	<span>📄</span> <span>↓</span>

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## Latest reports

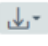
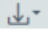
Recording	Organization	Updated	Approval	
● PID023d3	Sleep test center	16.12.2021 09:46 Not assigned	16.12.2021 09:48	 
● PID0a74a	Sleep test center	15.12.2021 15:16 Not assigned	16.12.2021 09:48	 

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**Download**

Recording data

-  Recording data 1 day(s)
-  TC58807\_Diagnostic\_Report\_16\_12\_2021.pdf
- Other documents
-  recording-information.pdf

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3 file(s)

+ Add file or drag here

<input type="checkbox"/>	Filename	Added			
<input type="checkbox"/>	recording-information.pdf				
<input type="checkbox"/>	Document_1.txt	19.12.2021 14:53			

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Navigation: < Back | Bittium | Dashboard | Recordings | EN | Sales Technician

Patient ID: PID0a870\_013696 Bittium HSAT

Organization	Sleep test center	Created	21.12.2021 13:38	Last data transfer	21.12.2021 13:39
Status	● Ended	Started	23.05.2021 22:53	Last update	23.12.2021 13:41
Assignment	None	Ready for analysis	21.12.2021 13:42	Duration	1 / 1 day(s)

Recording data | Patient details | Anamneses | Patient diary | Recording details | Attachments

Download  Include all attachments Cancel Save

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Recording data						
<input type="checkbox"/>	Day No.	Start date of file	Duration	Status	Quality	Notes
<input type="checkbox"/>	1/1	23.05.2021 19:53	08h 00m	● Open	Poor	Add notes

Navigation: < Back | Bittium | Dashboard | Recordings | EN | Sales Technician

Patient ID: PID020c5\_005914 Bittium HSAT

Organization	Evismo test center	Created	07.09.2021 15:04	Last data transfer	No date
Status	○ Rejected	Started	No date	Last update	22.09.2021 14:37
Assignment	None	Ready for analysis	No date	Duration	

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## Reject recording

Patient ID: **PID023d3**

The recorder 0000780 is currently in use in the selected recording. It will be released when the recording is rejected. Are you sure you want to reject the recording?

**This action cannot be undone.**

Why do you want to reject the recording?

- Corrupted recording data
- Not enough analyzable data due to noisy signal or unattached recorder
- Not enough sleep to write a statement
- Other

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## Delete recording

Patient ID: **PID023d3**

The recorder 0000780 is currently in use in the selected recording. It will be released when the recording is deleted. Are you sure you want to delete the recording?

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Bittium									
Dashboard		Recordings		EN		Sales Technician		Technician	
Recordings									
<div style="border: 1px solid red; padding: 5px;"> <ul style="list-style-type: none"> <li>• Not enough analyzable data due to noisy signal or unattached recorder</li> <li>• Not enough sleep during the recording to write the statement</li> </ul> </div>									
Patient ID	Search	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
DataTransferTest	○ Rejected	20.12.2021 14:58	21.12.2021 15:11	Sleep test center	20.12.2021 14:06	Bittium HSAT Plus	0000516	None	↓

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Bittium									
Dashboard		Recordings		EN		Sales Technician		Technician	
Patient ID: <b>PID023d3</b> Sleep test Bittium HSAT									
<div style="border: 1px solid red; padding: 5px;"> <ul style="list-style-type: none"> <li>• Not enough analyzable data due to noisy signal or unattached recorder</li> <li>• Not enough sleep during the recording to write the statement</li> </ul> </div>									
Organization	Created	20.12.2021 14:06		Last data transfer	20.12.2021 14:58				
Status	Started	20.12.2021 14:09		Last update	21.12.2021 15:11				

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The screenshot shows the Bittium patient record for Patient ID: PID0a74a (Bittium HSAT). The interface includes a navigation bar with 'Back', 'Bittium', 'Dashboard', and 'Recordings'. The user is logged in as 'Sales Technician'. The patient's status is 'Closed'. Below this, there are tabs for 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. A 'Download' dialog is open, with a red box highlighting the 'Download' button and the 'Include all attachments' checkbox. The 'Recording data' table below shows two entries for recording nights. The second entry, for night 2/2 on 17.12.2021, is selected with a checkmark in a red box.

Organization	Sleep test center	Created	16.12.2021 15:27	Last data transfer	16.12.2021 16:24
Status	Closed	Started	16.12.2021 15:36	Last update	21.12.2021 13:58
Assignment	None	Ready for analysis	17.12.2021 14:24	Duration	2 / 2 day(s)

Recording data Patient details Anamneses Patient diary Recording details Attachments

Download  
 Include all attachments

Cancel Save

Day No.	Start date of file	Duration	Status	Quality	Notes
1/2	16.12.2021 15:36	00h 29m	Open	Poor	Add notes
2/2	17.12.2021 10:01	03h 00m	Open	Poor	Add notes

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## HERSTELLER

### Bittium MedicalSuite Service Plattform

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Copyright 2024 Bittium Biosignals Ltd. Alle Rechte vorbehalten. Die in diesem Dokument enthaltenen Informationen können ohne Vorankündigung geändert werden. Bittium Biosignals Ltd behält sich das Eigentum und alle anderen Rechte an dem in diesem Dokument enthaltenen Material vor. Der Nachdruck des Inhalts dieses Dokuments ist ohne vorherige schriftliche Genehmigung durch Bittium Biosignals Ltd untersagt.